

Notes:

>Do not use the rollator on bad weather(rain,storm,snow etc.)

>Do not close to fire

>Do not use on the sand & soil ground or other bad road condition.

>Please do not use this rollator on follow situation :

~the rush road ~large passenger flow public place~
road without light at night

>Please read manual and the notes on the rollator before

use


- > Do not use the rollator on the slope or rugged ground.
- >The elderly and child must be watched by guardian.
- >Do not put a baby on any part of the rollator.
- >Make sure children do not get hurt when open or folding the rollator.
- >Max load weight is 120 kg.
- >Do not sit any other part except sit plate.
- >Brake system is different with a car ,please do not rely on it too much.
- >When is brake do not push the rollator.
- >Please brake the rollator when you leave.
- >Before you use ,please confirm all the wheels are on the ground.
- >Do not folding the rollator when you use the shopping cart,when folding the rollator please caution your hands.
- >Please clean the soil and garbage on wheel after use.
- >Clean the frame by dry rag,keep it dry under the sunshine
- >Do not use chemical liquor to clean the rollator.
- >If long time no use please clean it and keep it on ventilative & dry place.

>Working temperature :-5℃——40℃

>Please keep the manual with the rollator (As gift also give the manual together)

>Please do not put the rollator under the blazing sun for a long time, do not put it near children and baby.

Final assembly diagram

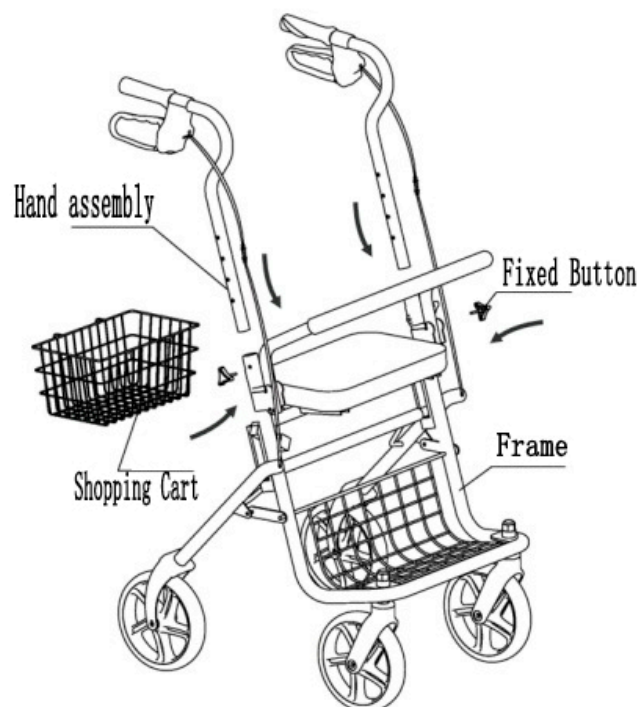

① Opening:

Take the rollator out from box, unlock the folding lock


To open the rollator(As the picture show),try the hand brake to make sure the brake is work,if not ,please adjust the brake adjust button! Put the back pipe get stuck on the main frame.



Note: When you take sit you must stop the rollator to keep safe , put your center of gravity on the center of seat plate!


② Braking:

>Depress left & right hand brake at the same time,you will hear a “KA” sound ,means the rear-wheel is stop.the rollator is braking.(Stop model! As follow picture)

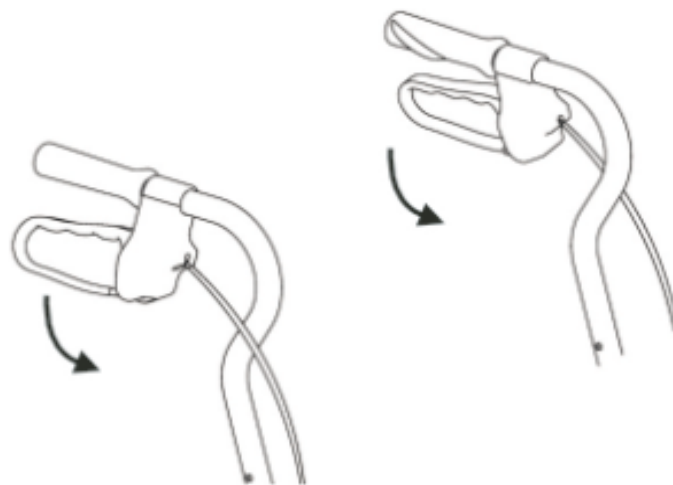


>Upward the hand brake a little bit to remove stop model ;Hang

on the hand brake at the same time to brake!(As follow)

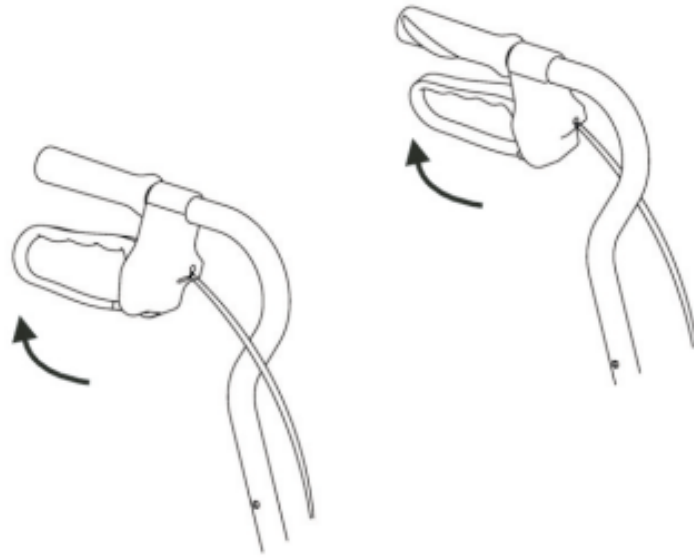

③ Install hand assembly & adjust the height of rollator.

>Unfold the rollator ,put the hand assembly into the main frame pipe, pull the hand assembly to align at the lashing eye ,use fixed button to adjust the height .

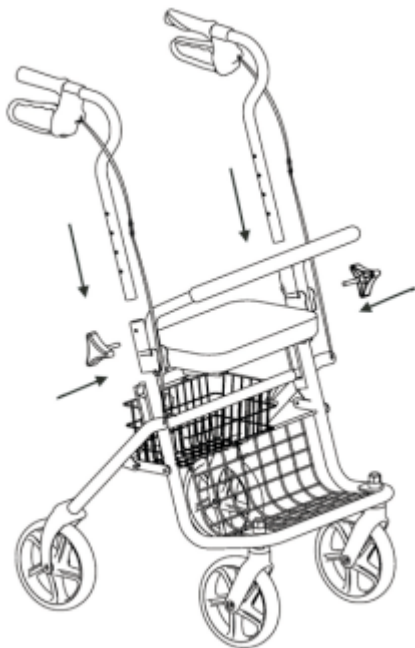

(Keep highly consistent)

④ Install the back pipe & shopping cart .


>Unfold the rollator , rotate the lock button to put up the back pipe,the fixed the back pipe on the main frame. Hang up the shopping cart on the main frame (please take off the shopping cart when you fold the rollator)



⑤ Adjust the brake system

If you hang on the hand brake but it can not work(not stop),you can adjust the brake adjust button.

>First adjust the rollator can move freely, screw the brake adjust button to adjustment (Around the same step for left&right)

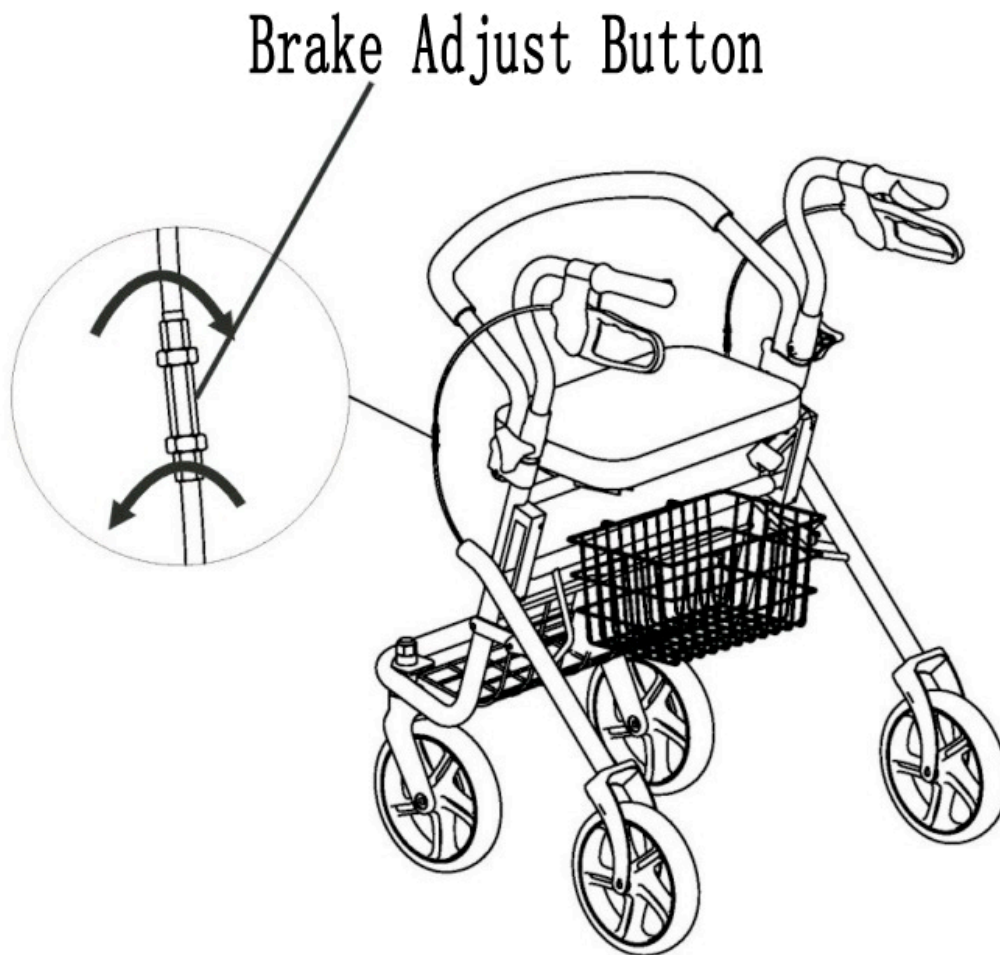

⑥ To folding

>Take off the shopping cart, Pull-up the back pipe ,unlock the lock button to fold . (As picture show) do not clamp the clothes or hand, it cannot stand after folding ,please keep the rollator on a safe place.



Specification

Property	Semi-automatic, Mechanical					
Material	Steel tube, Plastic, Leatherette					
Usage	Old people shopping, Rollator					
Temp Tange	(40°C --5°C)					
Standard						
	150-188cm	83-95 cm	56 cm	Seat Width 40 cm	Width unfolded 58cm	Overall length 63 cm

Safety Tips ! !

